


DT-100 DIGITAL THERMOMETER

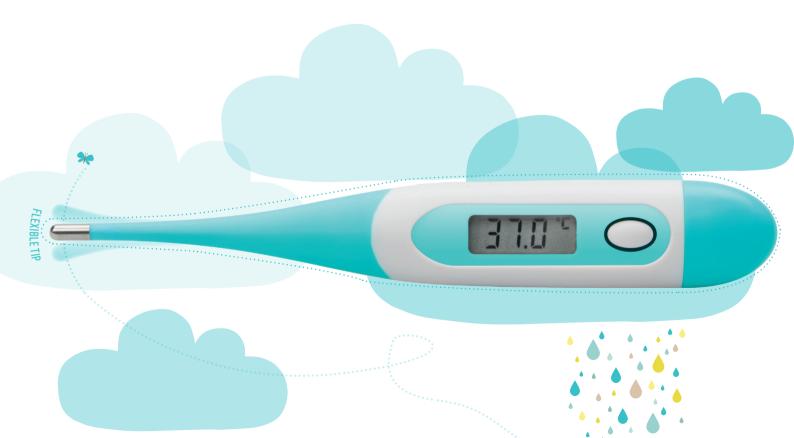

To find out whether your child has a **fever**, you must take their **temperature**. We often talk about a **fever** when the body temperature rises above 38°C. Body temperature varies, depending on the time of day.

Combining accuracy and ease of use, this DT-100 thermometer allows your baby's and infant's rectal, oral orunder-arm temperature to be taken. With its ultra-flexible tip, the "DT100" is comfortable and gentle on your baby.

Quick to use, it takes your temperature in just **10 seconds**. The thermometer lends itself well to measurements being taken at home and delights parents.



ADVICE:



- It is preferable to use an electronic rectal thermometer
- Lay your baby down on its back
- Coat the rubber tip with a cream which will help it slide in more smoothly (Vaseline or cream used when changing)
- Take your baby's temperature by gently inserting the thermometer into the rectum up to a depth of 2.5cm
- The thermometer will beep after a few seconds.
- Just remove it and read the temperature displayed.

It's the most reliable and least onerous method of taking your baby's temperature.



FEATURES:



Flexible tip



Unit of measurement in °C and °F



3 modes possible for taking a temperature: rectal, under the arm and oral



Measurement process takes less than 10 seconds



Fever alert at a temperature above 37.8°C



Last measurement stored in the memory



Waterproof




Auto







The best time to take your baby s temperature is in the morning and at the end of the day. However do not take its temperature after meals, bathtime or afternoon naps as these activities cause changes to body temperature. Wait for 20 minutes before starting.

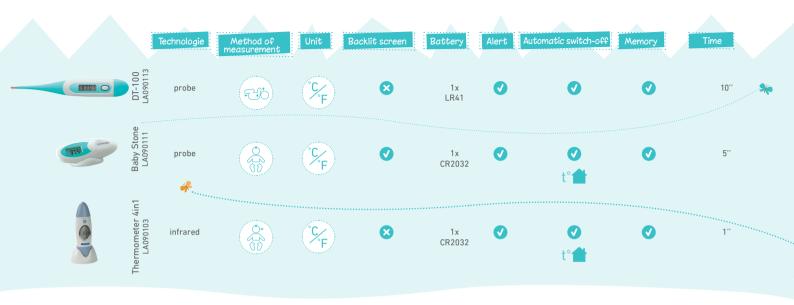









12 kg





















