LANAFORM





HEALTH

Baby Nose Vacuum

NASAL ASPIRATOR · ASPIRATEUR NASAL · NEUSREINIGER · NASENSAUGER · ASPIRADOR NASAL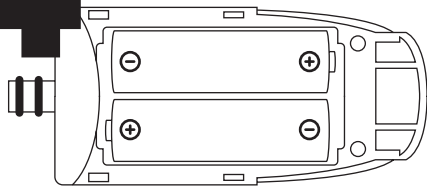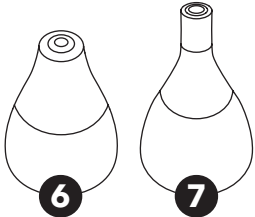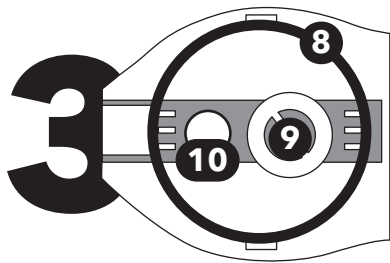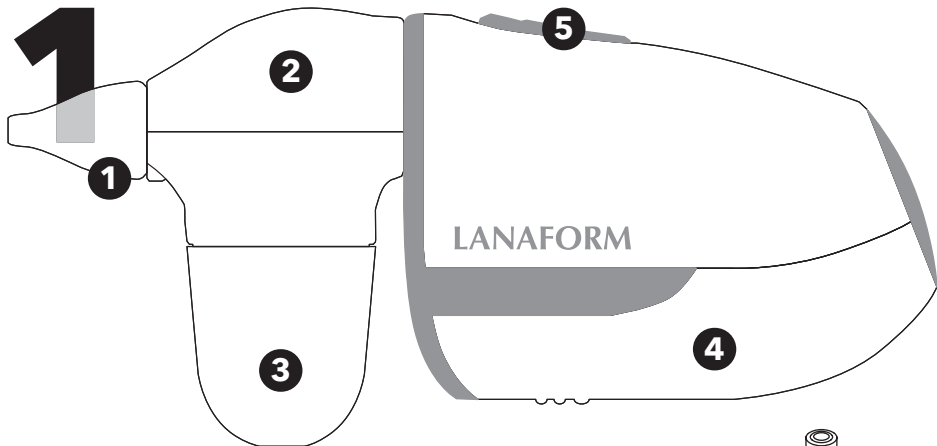

Precautionary measures

⚠ Please read all the instructions before using your product, particularly these basic safety instructions.

☒ The photographs and other product images in these instructions and on the packaging are as accurate as possible but do not claim to be a perfect reproduction of the product.

⚠ Danger

The nasal aspirator is designed solely for removing nasal secretions in children (0 to 12 years old). **Do not use the nasal aspirator for other body orifices or on animals. The nasal aspirator should be used only by adults**

able to follow the instructions for using the device in this instruction handbook. **Keep the nasal aspirator and its packaging out of the reach of children. This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision. The nasal aspirator is intended for strictly**

personal use in the home and is not suitable for use for medical or commercial purposes. **U**se of the device is not intended to be a substitute for a medical consultation and treatment. **T**his is why you should always consult your doctor first if you are in pain or ill! **D**o not use the device if your baby/infant has the following symptoms:

- Nose bleed
- Acute head, face or neck injury
- Bleeding disorder or blood disease

Please read the instructions carefully before using the device. **I**mproper use of the device can cause the following complications and risks:

- Discomfort and pain
- Irritation/nose bleed
- Bleeding in the mucous membranes

Consult a doctor if the user has a nasal infection. **G**ently place the silicon nozzle of the device in the nostril before treating your baby/infant. **I**f your baby or infant does not feel well when cleaning their nose, stop using the device immediately and consult your doctor. **W**hen in use, make sure your baby does not hold their breath, close their mouth or swallow their saliva. **T**his could create sub-pressure which can damage the eardrum. **P**lease observe general measures on hygiene when using the device. **C**lean the silicon nozzle before each use.

⚠ Warning

Do not use accessories that are not recommended by Lanaform or that are not supplied with this device. **P**lease use only those suction nozzles supplied with the nasal aspirator. **N**o other nozzle should be used with the device. **A**lways keep the device away from heat sources such as stoves, radiators, etc. **D**o not expose the nasal aspirator to extremes of temperature (above 55°C or below -25°C), to a humidity level that is too high (+85%) or to sunlight. **D**o not use the device if the room temperature is below 16°C or above 35°C or if the humidity level is above 85%. **D**o not use this device in a room in which aerosols (sprays) are used or in a room

in which oxygen is administered. **D**o not drop the device and do not use it if it is damaged. **I**f in doubt, please contact our customer service department or your reseller. **D**o not expose the nasal aspirator to electric shocks. **I**f there is a problem with your device, please contact your reseller. **D**o not try to repair the device yourself. **T**ake it back to your reseller. **D**o not wash the device with water or any other liquid. **F**or further information, consult the maintenance instructions in this manual.

Notes on batteries

It is recommended that the batteries be removed from the device whenever you are not using it for a prolonged period of time. **R**eplace the batteries whenever

the power and/or suction noise weaken when the device is being used. **Always** replace all the batteries at the same time. **If** the liquid of the battery cell comes into contact with your skin or eyes, rinse the area affected with water and consult a doctor. **Children** might swallow the batteries and suffocate. **Keep** the batteries out of the reach of children! **Check** the plus (+) et minus (-) polarity symbols. **If** a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth. **Protect** the batteries from excessive heat. **Do not** throw batteries onto the fire. **Do not** attempt to re-charge or short-circuit the batteries. **Do not** use rechargeable batteries.

Contents of the packaging

- 1 baby nose vacuum
- 1 wide silicon nozzle
- 1 fine silicon nozzle
- 1 storage case
- 1 instruction handbook

Presentation

Thank you for choosing the Baby Nose Vacuum.

You can effectively and gently remove nasal secretions of infants, toddlers and children up to the age of 12 years old with this nasal aspirator.

The mouthpiece and the secretions container are very conveniently removable for easy cleaning. The baby nose vacuum comes with 2 reusable ergonomic nozzles and a storage case. Suction power limited to the value recommended by paediatricians of 60 kPa ensures that the Baby Nose Vacuum can be used safely. All these features mean that it can be used every day in the long term.

Please note that the entire main unit must be replaced (including the silicon nozzles and the nasal secretions container) when the device has been used by several people. This is why we recommend changing the entire main unit after it has been in use for a year.

Description of the device

☒ 1 The device

- 01 Silicon nozzle
- 02 Main unit
- 03 Secretions container
- 04 Battery compartment
- 05 On/off button

☒ 2 Silicon nozzles

- 06 Wide nozzle
Suitable for removing solid nasal secretions
- 07 Fine nozzle
Suitable for removing liquid nasal secretions

☒ 3 Main unit

- 08 Silicon seal
- 09 Valve
- 10 Space for removing secretions

☒ 4 Installing the batteries

- 01 Open the battery compartment on the main unit of the device.
- 02 Insert two 1.5 AA batteries (not supplied), matching the polarities.
- 03 Replace the protective cover.

Use

The suction principle of the nasal aspirator works intermittently.


To set it going, proceed as follows:

- 01 Select the silicon nozzle you want.

 Wide silicon nozzle: suitable for removing solid nasal secretions

 Fine silicon nozzle: suitable for removing liquid nasal secretions


- 02 Put the silicon nozzle you have chosen on the tip of the device. Make sure that the silicon nozzle is placed as far as the recess on the tip of the main unit.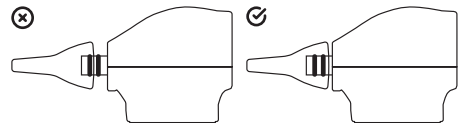

- 03 Press the On/Off button to check that the device is running properly before inserting it into the nose.
- 04 Hold or put the child to be treated in an upright position or tilted slightly backwards. Do not use the device when the child to be treated is lying down. Gently insert the tip of the silicon nozzle into the nostril or against the nasal septum.

- ⚠ **The tip of the silicon nozzle must not be fully inserted into the nose**

Hold the device during treatment at an angle of 20° to 60° to the face.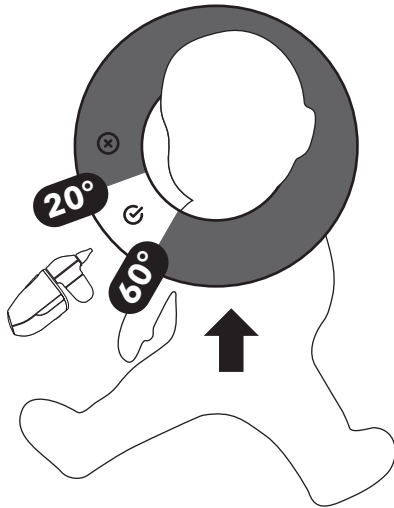

- 05 Press the On/Off button and suck up the nasal secretions.

- ⚠ **Make sure that nasal secretions do not get inside the valve.**

Repeat in the other nostril, if necessary. Stop using the device as soon as the nasal secretions container is filled to the "MAX" mark.


- ⚠ **If the child feels discomfort or pain, stop using it immediately.**

- 06 Release the On/Off button to stop using the device.


Maintenance and Care

For hygiene reasons and to ensure optimum operation, clean the device after each use.


- 01 Remove the used silicon nozzle and the main unit from the device.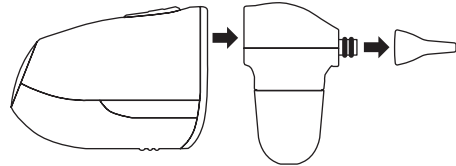

- 02 Turn the nasal secretions container anticlockwise (rotating it half way round). Then pull it down.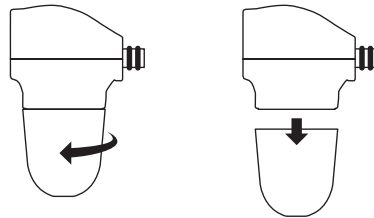

- 03 We also recommend that you remove the rubber seal (1) and the valve (2) from the main unit so that bacteria cannot settle there.

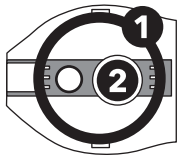- 04 To disinfect the silicon nozzle and the nasal secretions container, place them in boiling water for 3 to 5 minutes. Protect them from any contact with the hot bottom of the pan. Then place the different items on a dry, clean and absorbent surface, and let them dry completely (for at least 4 hours).
- 05 Clean the rubber seal and valve with hot water and a little mild soap. Do not use any abrasive cleaning products.
- ⚠ Using a dirty or damaged valve can reduce the nasal aspirator's suction power. The valve must be replaced as soon as you notice any dirt or damage. In addition, the valve must dry completely after cleaning in order to allow the device to run properly. Even with low residual humidity, suction power is reduced or even zero.**

- 06 Clean the main unit with water only. Do not use soap or chemical cleaning products because they could block the tip of the main unit.
- 07 Use a clean, dry cloth to clean the battery compartment. Do not use cleaning products that are irritants or solvents. The compartment must not be cleaned with water or be immersed in water.
- 08 Carefully dry all the items with a soft cloth. Only reassemble the items when are completely dry.
- 09 Keep the device in its storage case away from sunlight, dust and moisture.

Technical data

- Name and model: Baby Nose Vacuum LA131103
- Suction power: 450 mmHg (60kPa)
- Sound level: <75dB
- Tension: 3V (2 x 1.5V AA)
- Type of battery: 2 x AA (not included)
- Product dimensions: 15.4 x 4.2 x 7 cm
- Weight: 127 g (without batteries)

Material:

- Casing and main unit: ABS
- Nasal secretions container: PC
- Nozzles: silicon

Conditions of use:

- Temperature: 16°C to 35°C (61°F to 95°F)
- Moisture content of the air <85%

Storage and transport conditions:

- Temperature: -25°C to +55°C (-13°F to 131°F)
- Moisture content of the air <85%

Advice regarding the disposal of waste

All the packaging is composed of materials that pose no hazard for the environment and which can be disposed of at your local sorting centre to be used as secondary raw materials. The cardboard may be disposed of in a paper recycling bin. The packaging film must be taken to your local sorting and recycling centre.

When you have finished using the device, please dispose of it in an environmentally friendly way and in accordance with the law.

Prior to disposal, please remove the battery and dispose of it at a collection point so that it can be recycled.

Under no circumstances should spent batteries be disposed of with household waste.

Limited warranty

LANAFORM guarantees this product against any material or manufacturing defect for a period of two years from the date of purchase, except in the circumstances described below.

The LANAFORM guarantee does not cover damage caused as a result of normal wear to this product. In addition, the guarantee covering this LANAFORM product does not cover damage caused by abusive or inappropriate or incorrect use, accidents, the use of unauthorized accessories, changes made to the product or any other circumstance, of whatever sort, that is outside LANAFORM's control.

LANAFORM may not be held liable for any type of circumstantial, indirect or specific damage.

All implicit guarantees relating to the suitability of the product are limited to a period of two years from the initial date of purchase as long as a copy of the proof of purchase can be supplied.

On receipt, LANAFORM will repair or replace your appliance at its discretion and will return it to you. The guarantee is only valid through LANAFORM's Service Centre. Any attempt to maintain this product by a person other than LANAFORM's Service Centre will render this guarantee void.

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