PRECAUTIONARY STATEMENT

▲ Please read all the instructions before using your product, particularly these basic safety instructions.

 The photographs and other images of the product in this manual and on the packaging are as accurate as possible, but do not claim to be a perfect reproduction of the product.

▲ Danger

This pillow is not intended for use by people, including children, with limited physical, sensory or mental abilities, or by people lacking experience or knowledge, unless they are supervised or have been instructed beforehand on how to use the pillow by someone responsible for their safety. Children should be supervised to make sure they do not play with the pillow.

▲ Warning

This pillow should not be used by young children. · If you are concerned about your health, consult a doctor before using this pillow. · If you experience any pain when using this pillow, stop using it immediately and consult your doctor.

▲ Precautions

Use this pillow only according to the instructions given in this manual. · Do not use accessories other than those recommended by LANAFORM or accessories which are not supplied with this pillow. · Do not use safety pins, do not scratch the pillow with a sharp object and do not beat it with a blunt object. · Never insert any object into the pillow. · Use cold water (maximum 38°C/100°F). · Do not screw or unscrew the reservoir plug by hand (see instructions for use). In fact, the seal located at the bottom of the reservoir might be incorrectly positioned and cause water to leak.

INTRODUCTION

Thank you for choosing the Aqua Comfort by LANAFORM. The Aqua Comfort instantly adjusts to your position when you are asleep. What's more, you can choose the firmness you want by changing the volume of water in the pillow's reservoir.

CONTENTS OF THE PACKAGING

- 1 Aqua Comfort
- 1 Water spout

INSTRUCTIONS FOR USE  1 & 2

- 1 Keep the pillow vertical and unscrew the water reservoir plug using the water spout **ONLY**.
- 2 Remove the plug of the water spout.

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- Screw the water spout onto the pillow.
- To fill the Aqua Comfort, hold it using your thumb and index finger. Pour the volume of water you want into the pillow.

Level of firmness	Quantity of water	
	Litres	Ounces
Soft	2L	±70 oz
Medium	3L	±100 oz
Firm	4-5L	±135 oz

ⓘ As an indication: we suggest you start mid-level and then add or remove the water as you wish.

- Always holding the pillow vertically, remove the air contained in the reservoir by exerting pressure on the area above the water level. **3 & 4**
- Unscrew the water spout from the pillow.
- Replace the reservoir plug **ONLY** by using the water spout. Turn the water spout clockwise as far as you can.
- Check that the plug is properly closed to ensure the reservoir is completely sealed.

CLEANING

We recommend emptying the water reservoir every 3 months.

Wash your pillow by hand and in cold water. Do not use bleach.

Your pillow can be dry-cleaned, but not cleaned in the washing machine.

Please hang your pillow up to let it dry. Do not expose it to the sun or leave it to dry close to a heat source. Do not use a tumble-dryer and do not iron.

▲ To avoid having to wash your Aqua Comfort too often, cover it with a pillowcase before using it.

STORAGE

If you are not using the pillow for a lengthy period, store it in a dry place. Check its condition carefully before using it again and its seal in particular.

⚠ BREAKDOWNS AND SOLUTIONS

ⓘ **The pillow is leaking.**

- ⓘ The reservoir plug has not been screwed on properly.
- ⓘ Unscrew the water reservoir plug and screw it back on using the water spout **ONLY**.

ⓘ **The pillow is too soft/firm.**

- ⓘ There is not enough/too much water in the water reservoir.
- ⓘ Follow the instructions for use and add/empty water until you reach the firmness required.

⚠ ADVICE REGARDING THE DISPOSAL OF WASTE

All the packaging is composed of materials that pose no hazard for the environment and which can be disposed of at your local sorting centre to be used as secondary raw materials. The cardboard may be disposed of in a paper recycling bin. The packaging film must be taken to your local sorting and recycling centre.

When you have finished using the device, please dispose of it in an environmentally friendly way and in accordance with the law.

LIMITED WARRANTY

LANAFORM guarantees this product against any material or manufacturing defect for a period of two years from the date of purchase, except in the circumstances described below.

The LANAFORM guarantee does not cover damage caused as a result of normal wear to this product. In addition, the guarantee covering this LANAFORM product does not cover damage caused by abusive or inappropriate or incorrect use, accidents, the use of unauthorized accessories, changes made to the product or any other circumstance, of whatever sort, that is outside LANAFORM's control.

LANAFORM may not be held liable for any type of circumstantial, indirect or specific damage.

All implicit guarantees relating to the suitability of the product are limited to a period of two years from the initial date of purchase as long as a copy of the proof of purchase can be supplied.

On receipt, LANAFORM will repair or replace your appliance at its discretion and will return it to you. The guarantee is only valid through LANAFORM's Service Centre. Any attempt to maintain this product by a person other than LANAFORM's Service Centre will render this guarantee void.

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